

---

## MediVET INSTRUCTIONS FOR USE & REPROCESSING

### Reusable Laryngoscope Blades and Handles

---

#### MediVET Laryngoscopes

MediVET Laryngoscopes consists of a handle which contains the batteries and detachable blades. Please read and follow these instructions for use of and keep them for future reference.

#### Intended Use

MediVET Laryngoscopes are used to examine the back of the throat, larynx, vocal folds, arytenoid cartilages and to pass an endotracheal tube.

#### Power Source

The Laryngoscope uses 2 x type-C cell batteries for power. Type-C cell batteries are similar to AA but thicker. Rechargeable batteries are slightly thicker and may not fit. To insert batteries, unscrew the bottom lid by rotating it anti-clockwise. Insert 2 x type-C cell batteries as shown in the picture.

#### Laryngoscope Assembly

Inspect the laryngoscope carefully for burrs, sharp edges, or other visually discernible defects.

Aligning the slot of the blade on to the hook pin of the handle and apply downward/forward force as shown in Step 1.

Apply force upward to bring the blade in operating position as shown in Step 2, it will click and locks the blade under the bar.


Verify the blade is lit properly.

Reverse the above instructions to remove the blade.

Note: Incorrectly mounting the blade may result in damage to blade or malfunction of device.

#### Light

Veterinary F/O Laryngoscopes have LED Fiber Optic illumination. F/O handle has an LED light source;

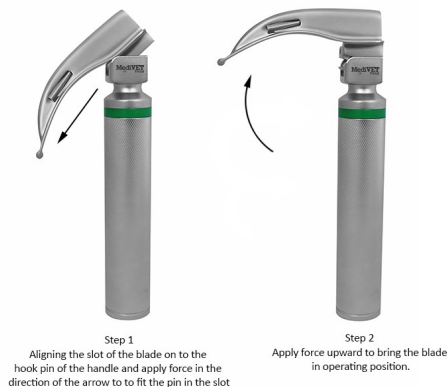

#### Maintenance and Service

Laryngoscopes do not require maintenance or service.

**⚠ WARNING** Conventional blade with Halogen bulb may become hot during use (risk of burn).

#### ⚠ CAUTION

Ensure the LED clean and free of any fingerprints after assembly. Clean the LED glass with a soft cloth or cotton ball moistened in alcohol if necessary.

Halogen lamps are pressurized to provide maximum illumination. Protect lamp's glass against abrasion and scratches.

Ensure that the batteries are inserted with correct polarity in the Laryngoscope Handle.

#### ⚠ WARNING

Only trained personnel should use a laryngoscope for intubation. Conventional blades & handles and green system blades & handles are not interchangeable with each other.

Remove the batteries if Laryngoscope is not in use for a longer period of time.

If handle becomes heat-up after inserting the batteries, remove the batteries immediately, and do not use the Laryngoscope.

Check the correct operation of the device before use.

Do not use the device if there are visible signs of damage or the light is flashing.

MediVET-Tools  
13 Fawkner Lane,  
Craigieburn VIC 3064  
Australia

E: [info@medivet-tools.com](mailto:info@medivet-tools.com)

W: <https://medivet-tools.com>