

MediVET Fiber Optic Oscopes User Manual

MediVET Oscopes

These instructions apply to the Veterinary F/O Mini Oscope.

Please read and follow these instructions for use of and keep them for future reference.

Intended Use

MediVET otoscopes are intended for use in the Veterinary market only. The otoscopes can be used for the examination of the ear, ear canal, and the tympanic membrane.

Power Source

The F/O otoscope uses 2 x AA cell batteries for power. Rechargeable batteries are not recommended. Rechargeable batteries are thicker than normal batteries and may not fit in the otoscope. To insert batteries, unscrew the bottom lid by rotating it anti-clockwise. Insert 2 x AA cell batteries as shown in the picture.

Magnification Lens

Otoscope has a magnification lens. Please see product details for magnification power. Magnification lens can be swivelled to both sides.

Light

Veterinary F/O Mini Oscope Set has LED F/O illumination.

Switch

Otoscope has on/off switch.

Head Fitting

Otoscope head has screw fitting. To install the head of otoscope to the handle, Rotate the head clockwise. Similarly, to uninstall the head, rotate it anti-clockwise.

Setup Oscope

Make sure the switch is OFF. Insert the batteries. Turn the otoscope ON.

Usage

The otoscope should only be inserted into the ear canal when a speculum is fitted.

Connecting the Speculum

Place the speculum on the head and rotate it, a small ridge inside the speculum will fit into the slit of the otoscope head.

Changing the LED source

Each Oscope set is provided with a spare LED. Veterinary Fiber Optic Oscope Set has F/O illumination, its LED is located at the base of the head. Rotate the head anti-clockwise. The LED can be replaced by pulling it out by holding it from the narrow collar. New LED can be inserted by pushing. No screwing is required for F/O Oscope LED.

⚠ WARNING Turn OFF the otoscope, remove the batteries before changing the LED.


Maintenance and Service

Oscopes do not require maintenance or service.

Application duration

The otoscopes are intended for short term usage.

⚠ WARNING The light emitted from the Oscope is potentially hazardous. Longer direct exposure could cause ocular damage. Avoid using the otoscope without a speculum.

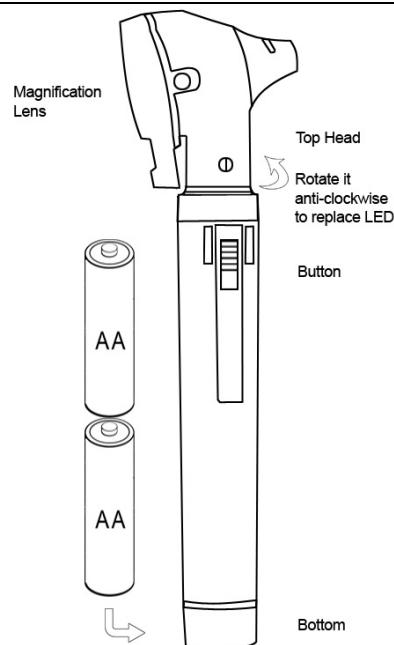

Cleaning and Disinfection

The Head may be cleaned and disinfected with an appropriate health care cleaner or disinfectant that has isopropyl alcohol or sodium hypochlorite (bleach) 1:10 as an active ingredient.

Do not sterilize the Head or the Handle.

Do not immerse Head in any solution.

Do not excessively soak the Head.

After disinfection inspect for visible signs of deterioration. If any signs of damage are visible. Discontinue use.

Reprocessing of Specula

Reprocessing of specula may be possible.

Instructions on hygienic reprocessing must be adhered to, based on national standards, laws and guidelines.

⚠ WARNING

Remove the batteries if otoscope is not in use for a longer period of time.

If handle becomes heat-up after inserting the batteries, remove the batteries immediately, and do not use the otoscope.

Check the correct operation of the device before use.

Do not use the device if there are visible signs of damage or the light is flashing.

MediVET-Tools
13 Fawkner Lane,
Craigieburn VIC 3064
Australia
E: info@medivet-tools.com
W: <https://medivet-tools.com>